Operation Manuel

****Product Model: PC-PE011****

****1. Features:****

This model was designed by Dr. Shi Shuzhong, a member of the Chinese Medical Association's Domestic Medical Society, Vice Chair of the Neonatology Group of the Pediatrics Society, and Chairman of the Neonatology Committee of the Shanghai Biomedical Engineering Society. The model is made from flexible PVC resin. To closely mimic clinical practice and teaching, the base is intentionally weighted, and the movable ring on the fixed frame is connected to the neck, allowing the model's head to tilt slightly backward when laid flat.

****2. Product Functions:****

1. Accurate anatomical representation of an infant's tongue, oral cavity, pharynx, epiglottis, larynx, vocal cords, and trachea.
2. Realistic newborn anatomical structure.
3. Endotracheal intubation via oral and nasal routes.
4. Simulation of functions using realistic materials.

****3. Instructions for Use:****

1. Stand at the side of the model's head. Tilt the head back 80 degrees so that the mouth, pharynx, and trachea are aligned, and open the mouth.
2. Before use, spray soap and detergent on the mouth, pharynx, and larynx.
3. With the left hand, hold the laryngoscope and insert the blade into the base of the tongue, lifting it upward. You will see the epiglottis; gradually move the blade to different areas of the epiglottis and continue to lift. The epiglottis will retract, making the tracheal opening clearly visible. With the right hand, take the endotracheal tube, lubricate it, and insert it into the trachea under direct visualization.

To check whether the endotracheal tube is properly inserted into the trachea, use one of the

following methods:

- a) Observe the semi-transparent tracheobronchial section of the model's head and neck cross-section to see the endotracheal tube.
- b) Hold the model upright and open the posterior pharyngeal wall through the rear hole to check if the tube is inserted into the trachea.
- c) Attach the endotracheal tube to the mask of a bag-mask resuscitator, apply positive pressure, and observe if the model's lungs expand, indicating the tube is in the trachea.

For nasal intubation, first spray lubricant into one nasal cavity, then insert the endotracheal tube into the nasal cavity, directing it backward and downward through the nasal passage to the pharynx. Under direct visualization with the laryngoscope, use forceps to guide the endotracheal tube into the trachea. The checking methods are the same as for oral intubation.

Note: The purpose of spraying lubricant on the mouth, pharynx, and larynx is to simulate the conditions of a real newborn during resuscitation and to reduce model wear and tear, extending its lifespan. Intubation should be performed with minimal force to avoid damage to the newborn or the model.

****4. Applications:****

This model is primarily used for teaching and training on endotracheal intubation during neonatal resuscitation. It is essential for medical schools, midwifery schools, nursing schools, hospitals, maternal and child health institutions, and other primary healthcare units. Mastery of intubation techniques using this model can standardize procedures and improve the survival rate of newborns during resuscitation.

****5. Maintenance and Warranty:****

(a) Part Replacement:

1. If the tongue or airway is damaged, first remove the simulated lungs, then unfasten the clips under the tongue. Place the model head in a vertical position and remove the tongue and airway through the rear hole.
2. Install a new tongue and airway by first applying lubricant to the trachea and pharynx. Insert the airway tube through the rear hole, aligning it with the pharynx, and secure it in place by snapping the clips back under the tongue. Finally, reassemble the lungs.

(b) Cleaning and Maintenance:

When not in use for extended periods, clean the model, pack it, and store it in a cool, dry place to extend its service life.

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